

SWISSFLEX[®]
e y e w e a r

like
wearing
nothing.



perfect fit
modular individuality
100% swiss-made



the better glasses fit, the better you see and feel!

Fitting components (nose, head, ear)

- All available parts in all available colors
- Identification of model and boxing size
- Mounting and demounting of temples
- Mounting and demounting of nose pads

Fitting System for nose

Modular ergonomic pads ensure best wearing comfort for each model and every face.


Nosepads regular -> multi fit						Slider for pad	
a1						A SFV slider for pad SFFA slider for pad SFCH slider for pad height-adjust SFCH slider for pad (CH-21)	
Pads with adapter -> adjustable fit							
a2							
	Mini pads with adapter -> exchangeable fit						
a3							
	Nosepad & pad with adapter baby -> riveted fit						
a4							
	Pads sport with adapter -> sporty fit						
a5	SF champ 11/12/17			SF champ 21			

Fitting System for head and ears


Modular ergonomic endpieces ensure best wearing comfort for each model and every face.

Model Systems	Temples	Endpieces			
		Stick			
b1 all models but SFLOb	straight 	SFC endpiece straight stick (fits SFC, SFCC, SFCK, SFCKC, SFLO, SFLOK, SFE, SFV) 1.1	SFAL endpiece straight stick (fits SFAL) 1.2	SFR endpiece straight stick (fits SFR, SFM) 1.3	
			Hook	Curved	Stick
b2 SF classic SF classic Contur SF classic Kid SF classic Kid Contur SF varius SF loop SF loop Kid	raster 	SFC endpiece raster hook 2.1	SFC endpiece raster hook strong 2.2	SFC endpiece raster curved 2.5	
		SFC endpiece raster hook kid 2.3	SFLOk endpiece raster hook gap 2.4		
b3 SF airlight	raster 	SFAL endpiece raster hook 3.1			
b4 SF eyephorics	raster 	SFE endpiece raster hook medium 4.1	SFE endpiece raster hook large 4.2	SFE endpiece raster curved 4.3	
b5 SF rainbow SF motion SF champ 21	raster 	SFR endpiece raster hook 5.1	SFR endpiece raster hook heavy 5.2	SFR endpiece raster curved 5.3	SFR endpiece raster stick 5.4
b6 SF fashion	raster 	SFFA endpiece raster hook 6.1			
b7 SF loop Baby	one piece frame 	SFLOb endpiece raster hook 7.1	SFLOb endpiece raster hook gap 7.2		


The fronts (only boxing size), temples, nose pads, earpieces bear specific identifications.

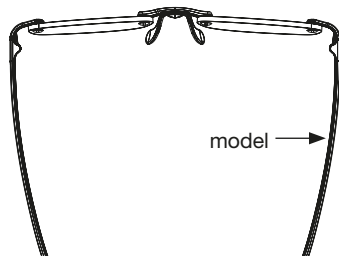


SFC, SFE, SFAL

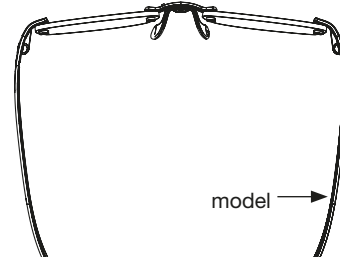


SFM

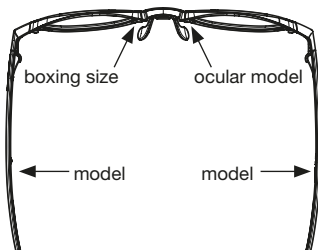

SFV

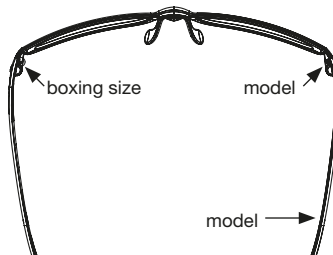


SFR

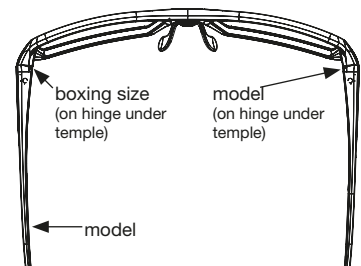


SFFA

Identification of Model and Boxing Size

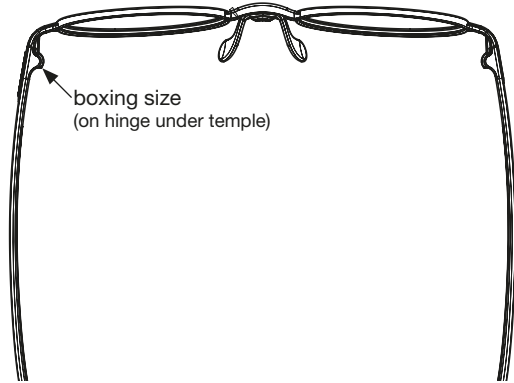


boxing size
(on hinge under temple)


SFLO, SFLOk

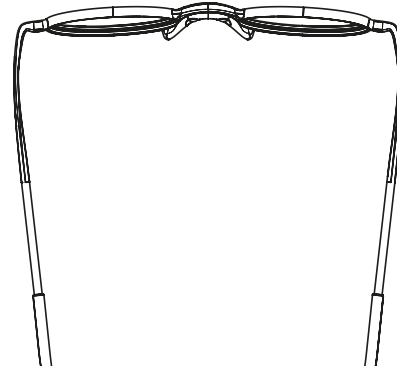


SFLOb > no identification

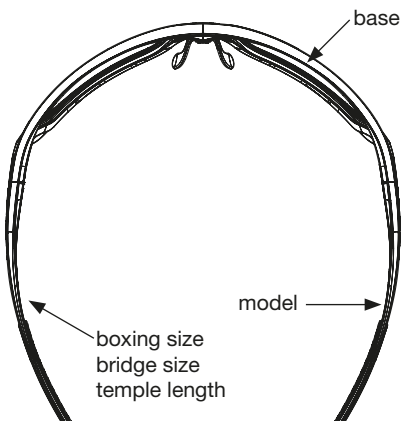


SFCH 11

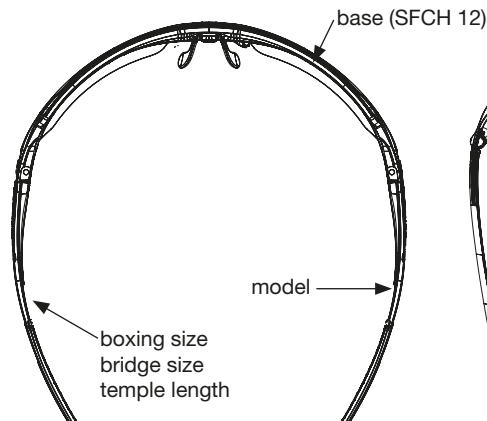


SFCH 12 / 17

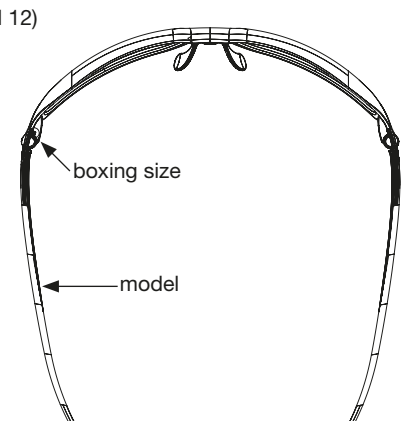

SFCH 21

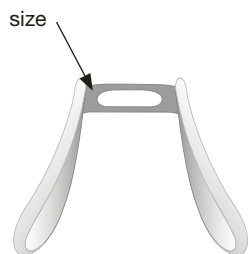

Nosepad

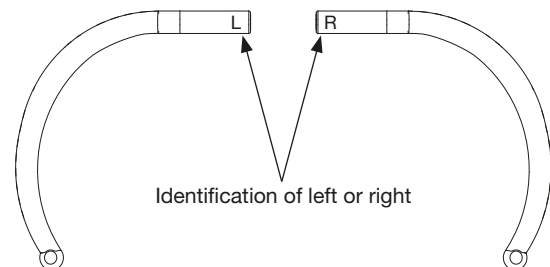

Identification of left or right

Earpieces

Mounting & Demounting of Temples

(for Opticians only)

General


Click-out

1. Check always opening direction of hinge
2. Hold frame at a vertical angle
3. Secure hinge firmly with finger
4. Remove temple with a jolt off the hinge or lever the temple off the hinge (mind opening angle!)

Click-in

1. Check always opening direction of hinge
2. Secure hinge firmly with finger
3. Click in the temple with light pressure

Methods

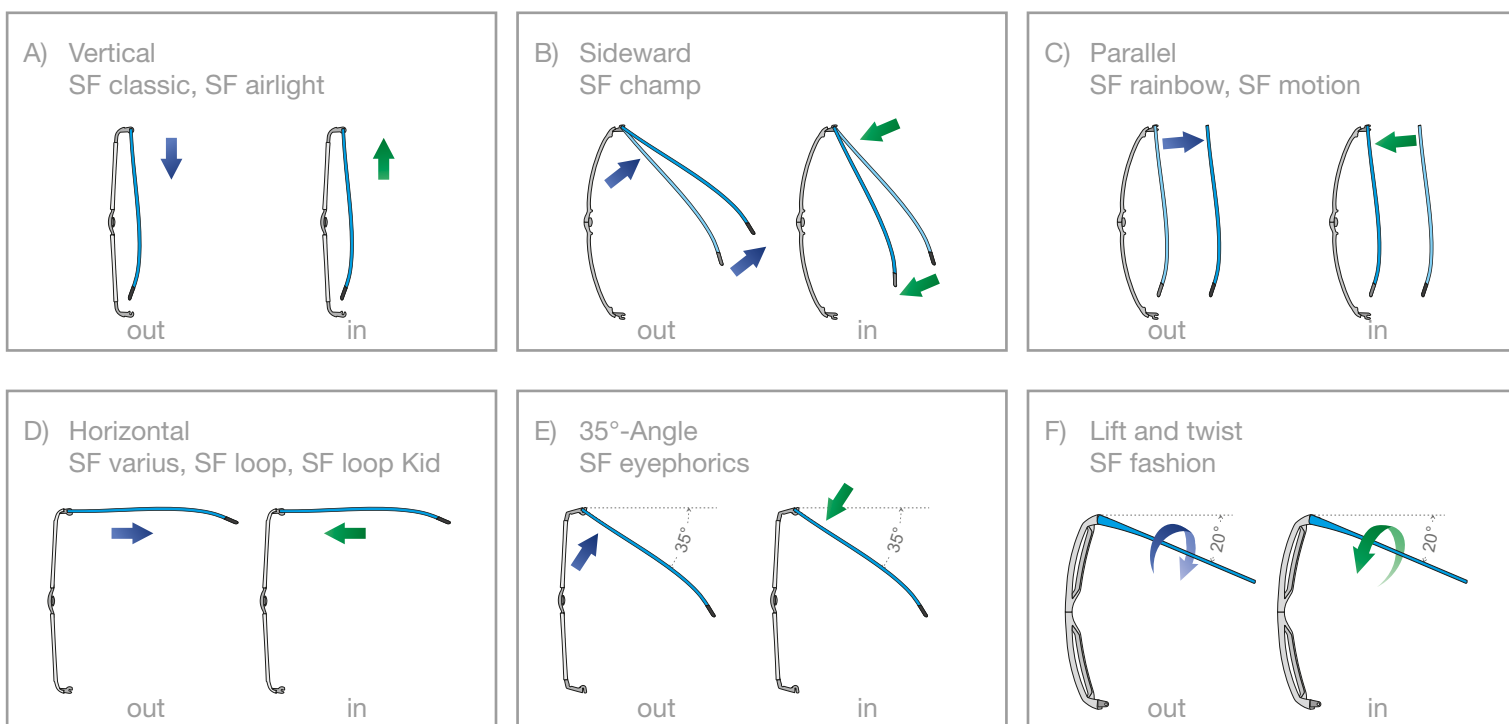

Particularities and Tips

- Mounting and demounting of temples is strong on purpose and serves the tight hold
- Protection of overexpansion for models SF loop, SF loop Kid, SF champ
- Removing temple with a jolt is important, otherwise the material can be overexpanded
- Older hinges may vary – please check opening direction of hinge
- Demonstration videos are provided in the login-area of our homepage

Mounting and demounting of temples should be performed by trained opticians only, not by end consumers.

Nose Pads / Mounting & Demounting (for Opticians only)


Nose Pads (all except SF varius)

Tightly clicking the nose pads in and out is necessary to ensure a tight fit on the center section.



Assembly (with tool)

1. Place nose pad (wings downwards)
2. Press with wedge into slot
3. Place center part with mushroom facing down on top of wedge
4. Press center part down vertically until nose pad clicks into place

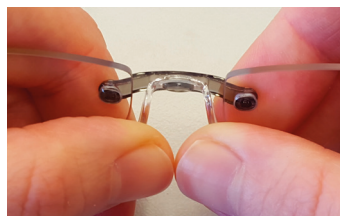


Disassembly (without tool)

1. Position middle part with nose pad inwards & downwards
2. Place thumb on lower part of ,wings'
3. Press ,wings' with both thumbs simultaneously between glasses

Nose Pads for SF varius

The nosepad is connected to the front with the slider - not with the regular ,mushroom'.

1. Slider System

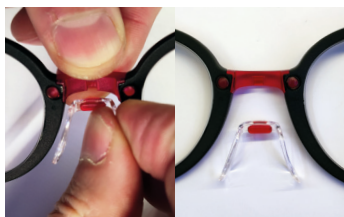


The nose pad including the slider can be easily pushed in or out with the thumb.


2. Mounting Slider to nosepad



1. The handle simplifies the insertion of the slider.

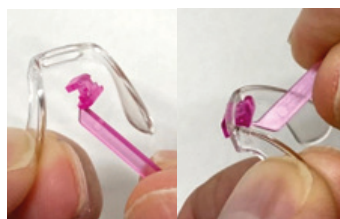

2. Insert the ,hook' through the wings in the hole.

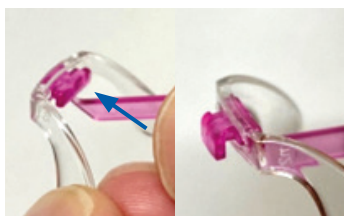

3. Ensure proper click-in of slider.



4. Turn the handle to detach it from the slider.

Note

- If handled correctly, the nose pad will not be damaged, when clicked in and out.
- Intentionally tight clicking in and out for a better grip.



www.swissflex-eyewear.com